



Insert

Using fluoroscopy, verify the femoral artery's suitability for arteriotomy closure with the EXOSEAL™ VCD, including the insertion angle (30-45 degrees) of the vascular sheath introducer.


1. Insert the EXOSEAL™ Device into the sheath, advance to the black marker band.
2. Retract the sheath back to the wirecowling.
3. Continue to retract the sheath compressing the green Indicator Wire Cowling against the white handle. A "click" will be heard.

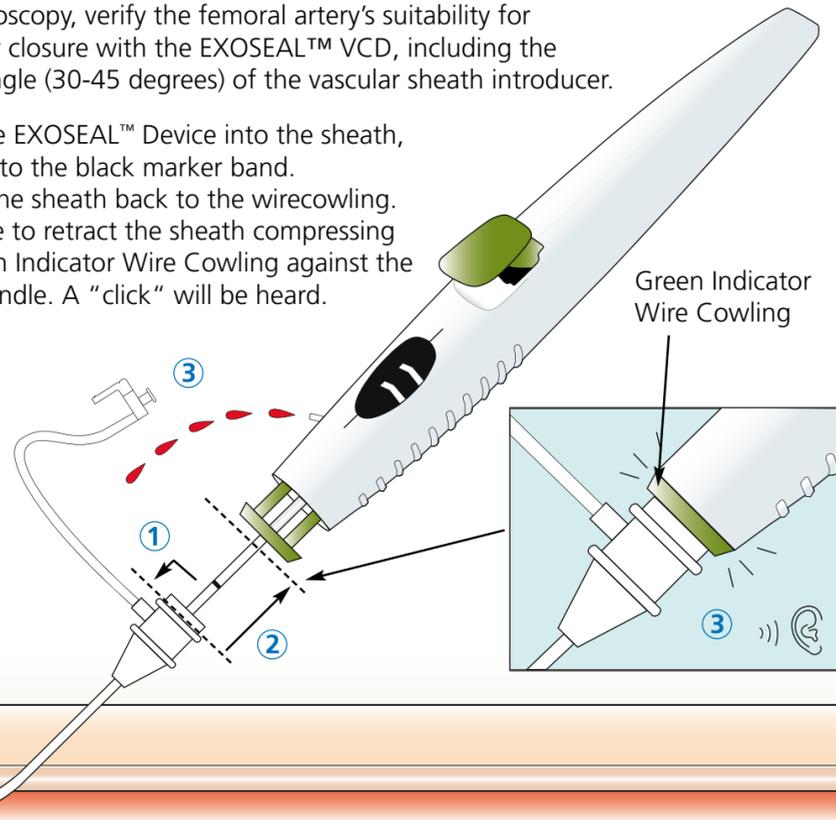

Retract

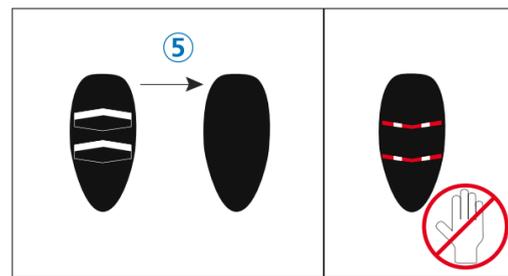

4. Retract the EXOSEAL™ Device and sheath together with the left hand, maintaining a 30°-45° angle with the right hand. Watch the bleed-back indicator for a significant reduction in pulsatile flow.
5. When the flow significantly changes and becomes non-pulsatile, slow your speed of retraction and watch for the indicator window to change to all BLACK.

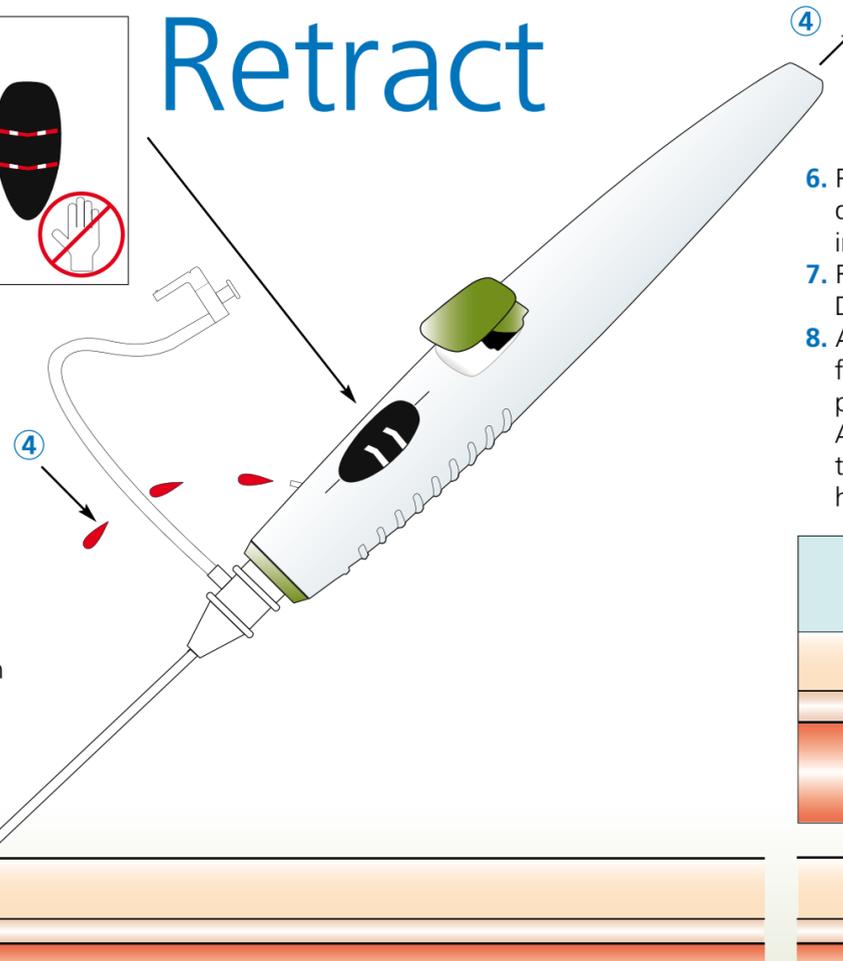

Deploy

6. Press the green plug deployment button fully into the white handle.
7. Remove the EXOSEAL™ Device and sheath together.
8. Apply light non-occlusive finger tip pressure at the puncture site for 2 mins. Apply a sterile dressing and then observe site as per hospital protocol.

